EN

Instructions for use of the medical device: Injectable hyaluronic acid-based implants Hyalual®

Description.

Hyalual® is an elastic gel of the hyaluronic acid of non-animal origin. The gel is sterile, pyrogen-free, and transparent. There is one pre-filled syringe of the implant Hyalual® in each package. Two sterile needles for each syringe are contained in the package. The volume of each syringe is indicated both on the cardboard package and on each syringe.

Formulation.

1.1% Injectable hyaluronic acid-based implant, the volume of which is 1 ml, contains: 11.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 1.0 ml.

1.1% Injectable hyaluronic acid-based implant, the volume of which is 2 ml, contains: 22.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 2.0 ml.

1.8% Injectable hyaluronic acid-based implant, the volume of which is 1 ml, contains: 18.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 1.0 ml.

1.8% Injectable hyaluronic acid-based implant, the volume of which is 2 ml, contains: 36.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 2.0 ml.

2.2% Injectable hyaluronic acid-based implant, the volume of which is 1 ml, contains: 22.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 1.0 ml.

2.2% Injectable hyaluronic acid-based implant, the volume of which is 2 ml, contains: 44.0 mg of Sodium hyaluronate; Succinate buffer, pH 7.4, to 2.0 ml.

Indications.

- for use in reconstructive surgery for filling such deformations as scars, for restoration of volumes caused by lipatrophy.
- for changes of the skin anatomy by restoring a lost volume and skin hydration, filling of wrinkles and skin cavities.

Mode of action.

Hyalual® is a biocompatible injectable implant with a prolonged action. Due to its specific visco-elastic properties it creates a natural homogenous and smooth volume for correction of the skin that has lost its tone, dehydrated skin areas, for filling wrinkles and cavities. A hyaluronic acid contained in Hyalual® provides a long-term absorption of the injectable implant by the skin tissue. This feature allows you to maintain an optimal level of hydration of the dermis, promotes a long-term skin elasticity. To achieve better results a patient may need to undergo one or more seances depending on recommendations of the expert conducting the procedure.

Contraindications.

Hyalual® is not to be used:

- for other forms of administration, except an intradermal one;
- in combination with peeling, laser or ultrasonic procedures;
- if a patient has skin diseases, inflammation, infected areas in the working area or area adjacent to it;
- if a patient is hypersensitive to the hyaluronic acid or has the history of allergic reactions and an anaphylactic shock;
- if a patient has autoimmune diseases;
- if a patient is pregnant and breastfeeding or a patient is a child;
- to administer in blood vessels;

- to administer in eyelid areas.

Since interaction with other known injectable products has not been investigated, it is not recommended to use Hyalual® together with other injectable drugs in the directly treated perimeter or area.

Dosage and administration.

Injection of Hyalual® is to be carried out by experts who know the technique of injectable implants administration. Before the procedure, you shall ask a patient in details about his/her previous illness and inform him about possible outcomes of action of the implant and possible side effects. Treated areas are to be thoroughly disinfected. It is recommended to inject an implant with needles contained in the package. Choice of the technique and number of procedures depends on the skin type of the patient, characteristics of the corrected defect and method of administration.

Hyalual® is injected into the middle layer of the dermis in the area of dehydration and/or wrinkles, cavities, which are to be corrected. If an implant is administered very deeply, that is into a deep layer of the dermis or subcutaneous tissue, correction will be less effective.

The volume of the injectable implant depends on the desired degree of correction. It is important not to do an excessive correction.

If you have a feeling of blockage or pressure during injection of the gel, stop injecting an implant and replace the needle.

Generally, 3-5 procedures are carried out spaced 1-2 weeks apart for optimal removal of visible defects of an aesthetic character. Performance of additional procedures ensures a long-term effect.

Side effects.

Like any implant intended to fill wrinkles and cavities, this medicine may cause side effects:

- allergic and skin manifestations: redness, swelling, dropsy, bruising, itching, slight pain in the area of injection may occur after administration and usually resolve within 72 hours;
- appearance of spots, loss of sensation in the area of administration of the implant;
- infection;

Rare cases of necrosis, formation of tuberculoma and granuloma, hypersensitivity and purulent inflammation appearing after injection of the hyaluronic acid are described in the literature.

An expert responsible for administration shall previously inform the patient about potential risks.

A customer should contact his doctor if side effects last more than one week or he/she needs additional information on other effects. The doctor is obliged to inform promptly about it the manufacturer or distributor.

Precautions for use.

Check the expiration date and integrity of the package before use. Do not use an implant if the expiration date has passed or the package is damaged.

Advise a patient not to take large doses of vitamin E, aspirin, antipyretics and anticoagulants for a week prior to the procedure. We recommend a client not to use makeup for 12 hours after injection and avoid sudden temperature changes (severe cold, sauna, steam room) within a week after the procedure. At the end of the procedure, dispose unused rests of the implant.

The product is intended for a single use. In case of re-use sterility can not be guaranteed, an efficiency level may be deteriorated and the gel can dry out, this complicates its administration through the needle.

Shelf life: 2 years.

Storage conditions.

Store in a dry, dark place, inaccessible to children at a temperature not above +30 °C. Freezing is not allowed.

Name and address of the manufacturer.

10, Amosova Street, Kyiv, 03680, Ukraine, "Yuriia Farm" Limited Liability Company, tel/fax +38(044) 275-92-42 at the plant: 108, Verbovetskogo Street, Cherkasy, 18030, Ukraine, "Yuriia Pharm" Limited Liability Company.

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